**Flaminal® Forte is indicated for moderately to highly exuding wounds.**

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## Flaminal® Forte

### Properties

Flaminal® Forte is a new class of wound healing agent, called enzyme alginogel®. It consists of an antibacterial enzyme system embedded in hydrated alginates. Flaminal® Forte supports the healing of a wound in a number of ways:

1. Flaminal® Forte keeps the wound moist.
2. Flaminal® Forte keeps the wound clean.
3. Flaminal® Forte offers antimicrobial protection.
4. Flaminal® Forte is safe for the skin and wound tissue.
5. Flaminal® Forte reduces wound odour caused by bacteria.
6. Flaminal® Forte helps to reduce excessive protease activity (protease = enzyme that breaks down proteins).
7. Flaminal® Forte reduces bacteria released from a biofilm.
8. Flaminal® Forte protects the wound border.

These are the prime properties for ensuring rapid wound healing.

Flaminal® Forte is indicated for moderately to highly exuding wounds such as:

- highly exuding 2nd degree burns (deep, superficial)
- highly exuding leg ulcers
- exuding pressure sores
- exuding oncology wounds
- pre-operative wound-bed preparation

### How to use

Applying or renewing Flaminal® Forte on a wound is easy and painless. Proceed as follows:

1. Rinsing and cleansing the wound and surrounding area is recommended prior to application of Flaminal® Forte.
2. Cover the entire wound bed with a sufficiently thick layer (e.g. 4-5 mm) of Flaminal® Forte
3. Cover Flaminal® Forte with a dressing. The choice of dressing should be based on the type of lesion:
  - Moderately exuding wound: transparent film (polyurethane) or paraffin gauze fixed by a non-adhesive bandage
  - Highly exuding: non-adherent dressing fixed by a non-adhesive bandage or by a hypoallergenic, adhesive tape
  - Very highly exuding: absorbent non-adherent compress fixed by a non-adhesive bandage or a hypoallergenic, adhesive tape

### Good to know

- Check the dressing daily. If it is leaking or if there is insufficient gel, Flaminal® Forte should be changed and a new dressing applied.
- During Flaminal® Forte treatment, dry whitish alginate flakes may appear on the wound border. They should not be removed. The flakes will prevent wound border maceration. Maceration slows the healing process of the wound.
- When dry flakes of alginates appear in the wound, the wound is too dry for Flaminal® Forte. Use Flaminal® Hydro. The flakes in the wound will disappear when the moisture balance is restored.
- The dressing with Flaminal® Forte can remain in place as long as the gel structure is intact (1 to 4 days, depending on the amount of exudate).
- In the initial phase of the treatment the wound may seem to become larger. This is a normal part of the healing process in which dead tissue is eliminated.
- The colour of the alginates may change slightly due to their natural origin. Colour changes have no effect on the quality of Flaminal® Forte.

### Special wounds

- Leg ulcers: adhesives often cause allergic reactions, particularly in patients who suffer from leg ulcers. Cover with a non-adhesive bandage.
- Undermined wounds: place sterile gauze tampons in the cavities as a support; apply a sufficient amount of Flaminal® Forte in the cavities between the tampons.
- Fistulas and small deep cavities: inject Flaminal® Forte carefully into the cavity using a dedicated cannula.
- Highly exuding wounds: apply a thicker layer of Flaminal® Forte.
- Slightly exuding wounds: apply Flaminal® Hydro.
- Exuding heel ulcers: apply Flaminal® Forte and cover with a V-shaped gauze compress.

### Precautions

Some wounds may indicate a complementary treatment. Consult a physician or wound specialist.

Flaminal® Forte can be used on infected wounds under medical supervision. Alginates have a faint odour. This odour has no influence on the healing process. Should there be a sudden distinct odour, consult a physician or wound specialist. Flaminal® Forte is not indicated in patients with a known allergy to one of the components (see under "Composition").

Flaminal® Forte cannot be used on the eyelids or in the eye. Should it come into contact with an eye, rinse the eye thoroughly with running water and consult a physician.

### Storage

Keep at room temperature (below 25°C) in a dry place in the original pack. Do not use Flaminal® Forte after the expiry date shown on the pack. The expiry date (USE BY: Year/Month) is the last day of the indicated month and year. Recap the tube or jar carefully immediately after use. Once opened, and if recapped carefully, a tube of Flaminal® Forte can be stored and used until the expiry date on the tube and box. A jar of Flaminal® Forte 500g however cannot be used more than one week after its first opening. We recommend that a pack of Flaminal® Forte be used for one patient only (single patient).

### Composition

Alginate (total alginate content 5.5% (g/g)); macrogol; enzyme system (glucose oxidase, lactoperoxidase, glucose, guaiacol); potassium sorbate; potassium iodide; buffer; purified water Ph.Eur.

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AT: Publilog GmbH, Neuhauserweg 16,
CH: Publicare AG, Vorderer Böde 9,
DE: Flen Health GmbH, Königsallee 92A,
UK: Flen Health Ltd, 1 Northumberland Avenue