Flamigel®

Indicaties

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Flamigel®

Indications

Flamigel® is a hydro-active colloid gel for the treatment of minor wounds which supports the fast healing of skin by covering the wound and creating an optimal moist healing environment. **Flamigel®** is used for post lasertherapy, on superficial burns (including those caused by radiotherapy) and on minor wounds (e.g. grazes, cuts). **Flamigel®** may also be used for superficial open wounds.

Properties

Flamigel® hydro-active colloid formulation can quickly help soothe the pain of minor wounds due to its cooling effect. **Flamigel®** helps to create optimal healing conditions to accelerate cell renewal, allowing the wound to heal fast and so

reduce the likelihood of scarring. **Flamigel®** also cushions the wound to provide a barrier against external contamination.

Flamigel® can be used for dry or wet wounds:

- In case of a dry wound **Flamigel®** turns the dry wound into a moist one ("hydrogel" effect). The dry wound absorbs the water contained in **Flamigel®** and promotes the dissolution of necrotic/damaged tissue.
- In case of a wet wound, the hydrocolloid of **Flamigel®** is activated and consequently **Flamigel®** absorbs excessive fluid from the wound and restores and maintains the water balance in the wound ("hydrocolloid" effect).

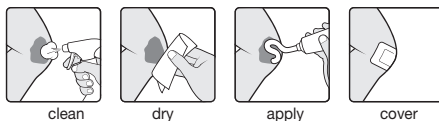
How to use

Wash hands before and after treating the wound. Clean and dry the wound gently before applying the gel.

For superficial burns and minor wounds (e.g. grazes, cuts): Carefully apply **Flamigel®** to the wound (without rubbing) once or twice per day (in case of sunburn apply twice to six times daily). If necessary, cover the wound with a plaster or dressing and change daily.

For superficial open wounds: Apply a thick layer of **Flamigel®** (5 mm) to the wound minimising contact with the wound edges. Cover **Flamigel®** with a sterile non-adhesive dressing and fix with gauze or a bandage. Leave **Flamigel®** in place as long as the gel structure is intact. When new skin cells start to form and the wound is closing, only apply a thin layer of **Flamigel®** to prevent weakening. If necessary, protect the wound edges.

For open wounds: Change daily.

Removing the gel:

- Wash hands before and after removing **Flamigel®**.
- **Flamigel®** can be removed simply and painlessly with wet gauze.

Precautions

- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use **Flamigel®** when there is a known allergic reaction to parabens or any of the ingredients.
- Do not use **Flamigel®** on the eyelids or in the eyes. If **Flamigel®** comes into contact with the eyes rinse the eyes with running water and consult a doctor.
- On extensive weeping or infected wounds, consult a doctor before using **Flamigel®** because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days seek medical advice.

Stop immediately using **Flamigel®** if irritation or allergy occurs.

If any serious incident occurs in relation to the device, or your condition worsens, please contact your doctor or pharmacist. This incident should be reported to Flen Health and the competent authorities of your country (the distributor, see below, can give you details of who this is). By reporting an incident, you can help provide more information on the safety and the efficacy of this medical device.

Storage conditions

- Store at room temperature (< 25°C), in a dry place and in the original packaging.
- Close the tube after use.
- Once opened, and if recapped carefully, a tube of **Flamigel®** can be stored and used for one year.
- Do not use **Flamigel®** after the expiry date shown on the pack. The expiry date (USE BY: Year/Month) is the last day of the indicated month and year.

Ingredients

Hydrocolloid, Arginine, Purified water, Macrogol, Branch Chained Fatty Acid (BCFA), Methyl-p-hydroxybenzoate (E218), Propyl-p-hydroxybenzoate (E216), Disodium EDTA.

How supplied

Sachets 4g and Tubes of 10g, 15g, 20g, 25g, 40g, 50g, 100g and 250g. Not all pack sizes may be marketed.

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NL: Flen Health BV, Postbus 5524,
4801 DZ Breda (The Netherlands)

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UK: Flen Health Ltd, 1 Northumberland Avenue,